



Understanding Cataract Surgery

Healthy Eye

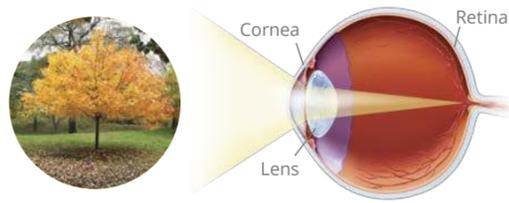

Eye With Cataract

A cataract is a lens clouded with proteins. Untreated, cataracts can completely block your vision.

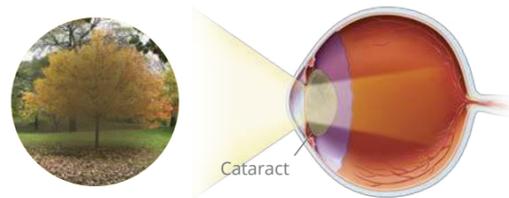


For more information about cataract surgery, please visit www.cataractsurgery.com.

What to Expect


1 Cataract Removal

Through a tiny incision, the surgeon removes the cloudy lens.

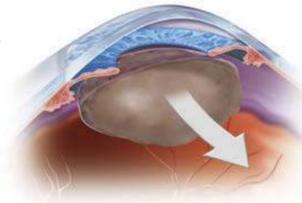


2 Lens Insertion

The surgeon injects and adjusts a flexible intraocular lens (IOL).

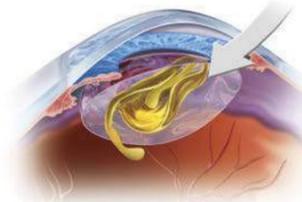

3 Vision Restored

The new lens allows you to see more clearly.

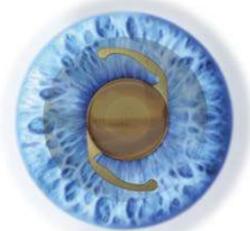

Choosing the IOL That Best Fits Your Everyday Lifestyle

	Near Vision	Intermediate Vision	Distance Vision	Astigmatism
MULTIFOCAL ASTIGMATISM-CORRECTING LENSES Designed to help reduce the need for glasses at all distances AcrySof® IQ ReSTOR® Toric IOLs: +3.0 D and +2.5 D with ACTIVEFOCUS™ Optical Design				
MULTIFOCAL LENSES Designed to help reduce the need for glasses at all distances AcrySof® IQ ReSTOR® IOLs: +3.0 D and +2.5 D with ACTIVEFOCUS™ Optical Design				
MONOFOCAL ASTIGMATISM-CORRECTING LENS Designed to help reduce the need for glasses at far distances; near and intermediate distances will require glasses AcrySof® IQ Toric IOL				
BASIC MONOFOCAL LENS Designed to help reduce the need for glasses at far distances; near and intermediate distances will require glasses AcrySof® IQ IOL				



IMPORTANT PRODUCT INFORMATION

AcrySof® IQ ReSTOR® Family of Multifocal IOLs

CAUTION: Restricted by law to sale by or on the order of a physician. **DESCRIPTION:** The AcrySof® IQ ReSTOR® multifocal Intraocular Lenses (IOLs) are artificial lenses implanted in the eye of adult patients following cataract surgery. These multifocal lenses (sometimes called “presbyopia-correcting IOLs”) are designed to allow for clear distance, intermediate, and near vision with the potential to be more independent of the need to use glasses for daily tasks. The AcrySof® IQ ReSTOR® Family includes the AcrySof® IQ ReSTOR® multifocal Toric IOLs which correct for astigmatism as well as cataracts. **WARNINGS / PRECAUTIONS:** You may experience and need to contact your eye doctor immediately if you have any of the following symptoms after cataract surgery: itching, redness, watering of your eye, sensitivity to light. The safety and effectiveness of AcrySof® IQ ReSTOR® IOLs have not been established in patients with eye conditions, such as an increase in eye pressure (glaucoma) or complications of diabetes in the eye (diabetic retinopathy). As with any surgical procedure, there are risks involved. These risks may include but are not limited to infection, damage to the lining of the cornea, the retinal layer which lines the inside back wall of your eye may become separated from the tissue next to it (retinal detachment), inflammation or swelling inside or outside the eye, damage to the iris (the colored diaphragm around the pupil), or an increase in eye pressure. With the AcrySof® IQ ReSTOR® multifocal IOLs, there may be a loss of sharpness of your vision that may become worse in dim light or in foggy conditions. There is also a possibility that you may have some visual effects such as rings or circles (halos) around lights at night. You may also have trouble seeing street signs due to bright lights or glare from oncoming headlights. **ATTENTION:** As with any surgical procedure, there are risks involved. Prior to surgery, ask your eye doctor to provide you with the Patient Information Brochure for the proposed AcrySof® IQ ReSTOR® multifocal IOL to be implanted. This document will further inform you of the risks and benefits associated with this IOL. Discuss any questions about possible risks and benefits with your eye doctor, your medical condition and any eye disease you may have.

AcrySof® IQ Toric IOL

CAUTION: Restricted by law to sale by or on the order of a physician. **DESCRIPTION:** The AcrySof® IQ Toric Intraocular Lenses (IOLs) are artificial lenses implanted in the eye of adult patients following cataract surgery. These lenses are designed to correct pre-existing corneal astigmatism, which is the inability of the eye to focus clearly at any distance because of difference curvatures on the cornea, and provide distance vision. **WARNINGS / PRECAUTIONS:** Contact your eye doctor immediately if you have any of the following symptoms while using the antibiotic eye drops prescribed by your doctor: itching, redness, watering of your eye, sensitivity to light. The safety and effectiveness of the AcrySof® IQ Toric IOL has not been established in patients with eye conditions, such as an increase in eye pressure (glaucoma) or complications of diabetes in the eye (diabetic retinopathy). As with any surgical procedure, there are risks involved. These risks may include, but are not limited to, infection, damage to the lining of the cornea, the retinal layer which lines the inside back wall of your eye may become separated from the tissue next to it (retinal detachment), inflammation or swelling inside or outside the eye, damage to the iris (the colored diaphragm around the pupil), an increase in eye pressure that cannot be controlled by medicine and secondary surgical procedure. A toric IOL corrects astigmatism only when it is placed in the correct position in the eye. There is a possibility that the toric IOL could be placed incorrectly or could move within the eye. This may result in less improvement or a reduction in vision because your astigmatism has not been fully corrected, or it may cause visual symptoms. **ATTENTION:** As with any surgical procedure, there are risks involved. Prior to surgery, ask your eye doctor to provide you with an AcrySof® IQ Toric Patient Information Brochure, which will inform you of the risks and benefits associated with this IOL. Discuss any questions about possible risks and benefits with your eye doctor.

AcrySof® IQ IOL

CAUTION: Restricted by law to sale by or on the order of a physician. **DESCRIPTION:** The AcrySof® IQ Intraocular Lenses (IOLs) are artificial lenses implanted in the eye of adult patients following cataract surgery. These lenses are designed to allow for clear distance vision. However, you will likely still need glasses for reading and for distance vision particularly if you already have astigmatism. **WARNINGS / PRECAUTIONS:** You may experience and need to contact your eye doctor immediately if you have any of the following symptoms while using the antibiotic eye drops prescribed by your doctor: itching, redness, watering of your eye, sensitivity to light. The safety and effectiveness of the AcrySof® IQ IOL has not been established in patients with certain eye conditions, such as an increase in eye pressure (glaucoma) or complications of diabetes in the eye (diabetic retinopathy). As with any surgical procedure, there are risks involved. These risks may include, but are not limited to, infection, damage to the ocular structures: lining (inner surface) of the cornea, damage to the iris (the colored diaphragm around the pupil), the retinal layer which lines the inside back wall of your eye may become separated from the tissue next to it (retinal detachment), inflammation or swelling inside or outside the eye, an increase in eye pressure that may not be controlled by medicine and secondary surgical procedure. There is a possibility that this IOL could be placed incorrectly or could move within the eye. This may result in less improvement or a reduction in vision, or it may cause visual symptoms. **ATTENTION:** As with any surgical procedure, there are risks involved. Prior to surgery, discuss any questions you may have about possible risks and benefits with your eye doctor.